operating instructions

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instructions

L A  B O Y[®]

Bronze Luxury-Lift[®]

Power recliner

Sillón reclinable

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la-z-boy.com

La-Z-Boy Incorporated
One La-Z-Boy Drive
Monroe, MI 48162

L A  B O Y[®]

Congratulations!

You are now the proud owner of a state-of-the-art Bronze Luxury-Lift® Power Recliner by La-Z-Boy.®

You have just selected one of the finest quality upholstered products produced in the furniture industry, backed by over 90 years of quality manufacturing experience.

La-Z-Boy is proud of our designs and craftsmanship and knows you will feel the same as you enjoy your new furniture's comfort and style.

IMPORTANT

Read the safety and other instructions before using your Bronze Luxury-Lift® Power Recliner. Keep this manual for future reference.

Table of Contents

Important Safety Instructions.....	4-5
Bronze Luxury-Lift® Power Recliner Assembly Instructions.....	6-9
Bronze Luxury-Lift® Power Recliner Glide Adjustment Instructions.....	10
Bronze Luxury-Lift® Power Recliner Battery Backup Assembly Instructions.....	11
Bronze Luxury-Lift® Power Recliner Operating Instructions.....	12-14
Bronze Luxury-Lift® Power Recliner Troubleshooting.....	16
Luxury-Lift® Hand Wand Removal Instructions.....	18
Luxury-Lift® Hand Wand Installation Instructions.....	19
Furniture Moving & Placement Instructions.....	20
Notes.....	21

Disclaimer: The information and products featured in this manual may be somewhat different from the product you purchased. La-Z-Boy reserves the right to make substitutions that are equal in quality and performance.

Important Safety Instructions:

Important: Read these instructions thoroughly before using.
When assembling, do not plug in until directed in Operating Instructions.

DANGER – To reduce the risk of electric shock, serious injury or death:

- Do not let children play on or near the chair when in use. Serious injury or death could occur if head or body parts are under the chair when the chair is in motion.
- Liquids exposed to electricity pose an immediate risk of electric shock. Do not use this unit if the power control is damaged, wet, in contact with or near spilled water or any liquid. Do not use the unit if the button activation switch has been exposed to water or any liquid. Do not handle the plug or remote control with wet hands.
- To prevent suffocation, keep plastic bags away from children.
- Keep cord out of the reach of children and pets that may cause damage.
- Do not attempt to disassemble or service the product, pump or motor; no serviceable parts are inside. Contact your La-Z-Boy® dealer for service.

WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- This product is intended for indoor residential use only.
- Keep upholstery away from flames or lit cigarettes. Upholstery may burn rapidly, with toxic gas and thick smoke. Fires that begin elsewhere in a home may spread and involve upholstery. For your safety, always use a properly installed smoke detector with working batteries.
- Unplug the power cord immediately if you detect strange odors or fumes coming from the unit.
- Plug directly into a wall outlet. Do not use an extension cord, adapter, or surge protector.
- Do not attempt to disconnect the power plug by pulling the power cord. Always hold the power plug and pull it out.
- Disconnect the electrical plug from outlet before cleaning, moving, servicing or when not in use for an extended period of time.
- Closely supervise children and disabled persons using this product.
- Remove hand wand when not in use, and store out of reach of children.
- This product has many moving parts, serious injury can occur if not used properly. Keep hands, fingers and body parts away from the mechanism. To reduce the risk of head entrapment and other injuries, keep children and pets away from openings and moving parts in and around furniture.
- Do not place power supply underneath the chair.
- Do not place cord under carpet or in a location where it could become worn.
- Do not allow electrical cord to be pinched anywhere.
- Keep the electrical cord away from heat sources.
- Do not use this product where the cord may be a trip hazard.
- Do not use this product if it is damaged. Do not use attachments or substitute parts.
- Never operate the unit with a damaged power supply, electrical cord, plug or connector. Check the condition of the power supplies, electrical cords, plugs and connectors often.
- Do not operate the unit until the back is installed and locked.
- Sit down in the unit slowly. To avoid tip over, do not throw your weight against the back of the unit.
- Only the occupant should operate the unit. Never attempt to open the legrest by pulling on it.

Save These Instructions

Important Safety Instructions:

Important: Read these instructions thoroughly before using.
When assembling, do not plug in until directed in Operating Instructions.

- To safely exit the unit and prevent a tripping hazard, bring the back upright and close the legrest.
- Unit is designed to ease exit from the chair. Take care to not slip out of chair when chair is in motion.
- Do not stand, sit or place extra weight on the arms, back or open legrest.
- Some medications and physical limitations may impair your ability to operate this product in a safe manner. Consult your physician if you are taking prescribed or over the counter medication, or if you have physical limitations.
- Users of medical devices should contact their health care provider before using this product. Do not use this product if you have poor circulation, are immobile, or cannot sense heat, except as directed by your health care provider.
- Do not use this product where aerosol (spray) products are being used or where oxygen is being administered.
- In the event of a power failure, the battery backup provided may not lift the user to a fully raised position. To ensure lift functionality during a power outage, a battery backup system must be purchased separately. Contact your La-Z-Boy® dealer for more information. Replace the power supply batteries every six months and after each power failure use.
- Before moving the lift chair, disconnect the electrical plug from the wall outlet. Secure the mechanism (as shown in the Operating Instructions) to prevent unexpected opening or movement. Moving the chair requires two people, one on each side of the chair, lifting only the lower arm frame near the base, never by the legrest or reclining back.
- This product is not intended to be used as a bed. Do not sleep in for an extended period.
- Double-insulated products are marked with the words “DOUBLE INSULATION” or “DOUBLE INSULATED” and may include the symbol M3  (square within a square). Do not attempt to ground these products, which use two systems of insulation instead of grounding. Double-insulated products should be serviced only by qualified service personnel, using replacement parts identical to the original parts.
- Burns may result from improper use and care of internal heating pads. To avoid risk of burns, do not use pins, do not allow heating pad area to become wet, and do not remove upholstery cover.
- Do not use pillow, blanket, or other cover between occupant and chair surface when heat is engaged.
- The Bronze lift chairs are intended for a single occupant, not exceeding 300 pounds. Do not operate the chair with more than one occupant in or on any part of the chair.
- If you give this product to someone else, include this Operating Instructions manual.

NOTICE – To reduce the risk of product damage:

- To prevent damage to the power supply, do not plug in until instructed.
- Never operate the unit except when it is on its base in the upright position.
- Never operate the unit until the shipping cable tie has been removed from the mechanism.
- After 2 minutes of continuous operation of the lift and recline features, allow the features to rest for 18 minutes.
- Gently press the buttons during operation and avoid hitting the buttons with pointed or hard objects.

Save These Instructions

Bronze Luxury-Lift® Power Recliner Assembly Instructions

NOTICE – To reduce the risk of product damage:

- Do not plug in until instructed.
- Never operate the unit until the shipping cable tie has been removed from the mechanism.

1. Choose a location for the lift chair that is close to an electrical outlet.
2. Carefully remove the chair from the carton and remove packaging materials. Lay the chair back aside.
3. Remove the red electrical components bag from the body bracket and unpack the contents; packed inside are the power supply and 120-volt power cord (FIGURE 1). Lay the contents aside.
4. Gently tip the chair body forward to rest on the front arms. Cut and remove the cable tie securing the mechanism assembly (FIGURE 2). Return chair body to its upright position.
5. Lay the back across the chair arms with the back cushion facing the seat, and turn the outside back cover over, toward the front of the chair (FIGURE 3).

WARNING – To reduce the risk of injury:

- Do not operate the unit until the back is installed and locked.


6. Install the back.
 - Note the location of the fastening tape sewn to the back cover; the mating fastening tape is on the bottom side of the rear rail (FIGURE 3). Do not attach the fastening tape at this time.
 - With the back across the chair arms, lift the back at a slight angle and align the brackets on the back with the brackets on the body (FIGURE 3).
 - Rotate and lower the back onto the body. Push down firmly on the top of the back to fully engage and lock both sets of brackets. An audible “snap” or “click” is heard from each side of the chair when the brackets are engaged and locked.
 - Lift the back upholstery cover and visually confirm that the back and body brackets are engaged on both sides (FIGURE 4). Then, using both hands, lift up on the chair back to ensure that it is locked in place. If one or both sides lift, the brackets are not fully engaged and locked. Remove the back and repeat the steps above. Refer to the last bullet for instructions on how to unlock a back bracket.
 - If you are unable to install the back properly, contact your La-Z-Boy® dealer for assistance.
 - **To Remove the Back:** Unplug the 120-volt power cord from the electrical outlet and disconnect the power supply from the motor connector (if connected). Loosen the fastening tape securing the outside back upholstery cover to the lower rear rail (if attached). Lift the back cover to locate the back brackets and locking tabs, one on each side (FIGURE 4). Press the tabs away from the brackets, toward the inside of the unit and lift up on the back to remove it from the body.

Never attempt to install or remove the back when the power supply is plugged into an electrical outlet or connected to the chair. Make sure the power supply is unplugged and disconnected from the chair before assembling or disassembling the chair.

(Assembly Instructions Continued)

Bronze Luxury-Lift® Power Recliner Assembly Instructions


FIGURE 1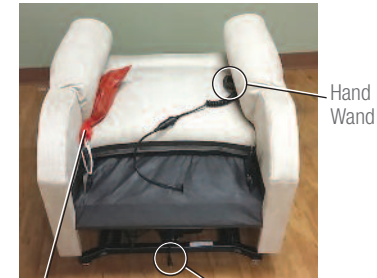

Electrical Components Bag
(Power Supply and
120-Volt Power Cord)


Motor Connector
at Rear Body

FIGURE 2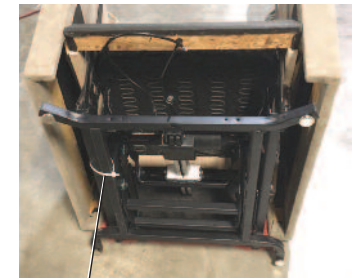

Shipping
Cable Tie

FIGURE 3

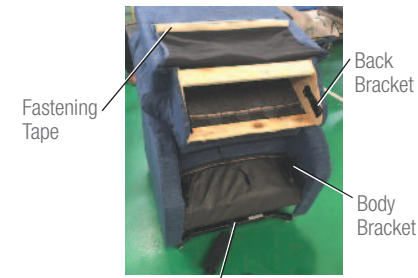Fastening
Tape

Back
Bracket

Body
Bracket

Fastening Tape

Note: Fastening tape located on bottom side of the body rear rail is not visible.

FIGURE 4

Back and Body
Brackets Engaged
and Locked

Locking Tab


Bronze Luxury-Lift® Power Recliner Assembly Instructions

7. Attach the outside back cover. The back tailgate upholstery cover attaches to the body using fastening tape. Pull the outside back upholstery cover down tight and firmly press the fastening tape on the outside back cover to the fastening tape on the bottom side of the rear rail. (FIGURE 5).
8. Make the power supply connections with the following steps.
 - Remove power supply and 120-volt power cord from box and unwind cord.
 - Connect the 2-prong pin and blade motor connector located at the back of the unit to the power supply. Align the motor connector to the power supply and push together until fully seated (FIGURE 6).
 - Connect the 120-volt power cord to the power supply. Align the 120-volt power cord connector to the power supply and push together until fully seated (FIGURE 6). Do not plug in until instructed in the *Operating Instructions*.
9. Carefully remove the hand wand from between the arm and seat. Remove the packing material and lay the hand wand across the seat, toward the front of the chair.
10. Proceed to the *Glide Adjustment Instructions*, followed by the *Battery Backup Assembly Instructions*.

(Assembly Instructions Continued)


Bronze Luxury-Lift® Power Recliner Assembly Instructions

FIGURE 5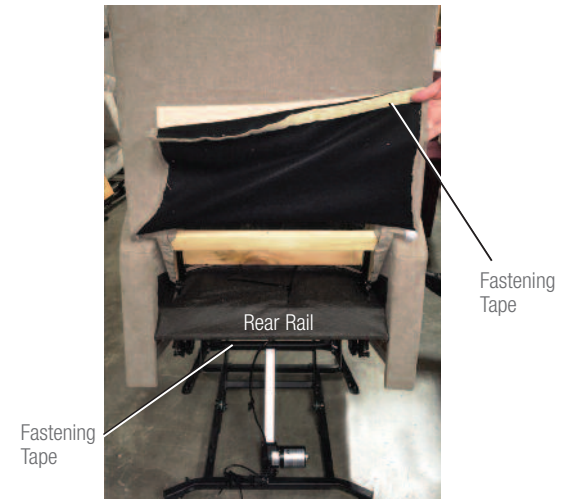

Note: Fastening tape located on bottom side of the body rear rail is not visible.

FIGURE 6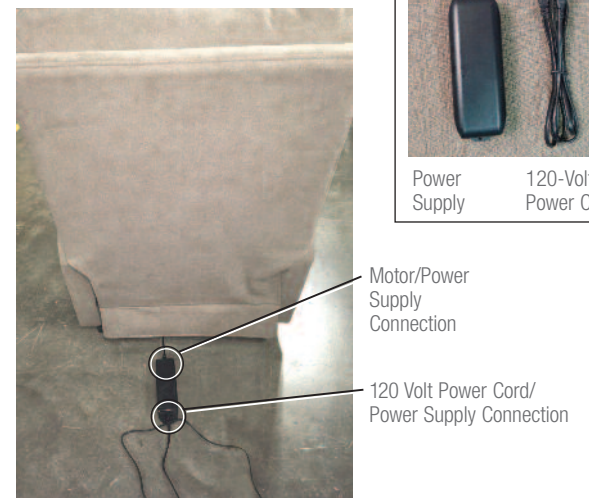

Bronze Luxury-Lift® Power Recliner Glide Adjustment Instructions


⚠ WARNING – To reduce the risk of serious injury:

- Provide a clear path for operation of the back and legrest. Place tables and area rugs at a distance to allow the legrest to fully extend without rubbing or interference.

Adjusting the Glides

Adjustable glides are located in bottom of the mechanism base assembly, one at each corner. Use the glides to raise, lower and level the chair as required. Make sure to adjust the glides to a height that provides a clear path for operation of the legrest (*see Furniture Placement on page 20*).

- Turn the glide clockwise to raise the chair or counterclockwise to lower the chair.
- Check alignment and make adjustments as necessary.
- Repeat these steps for each adjustable glide, if necessary.

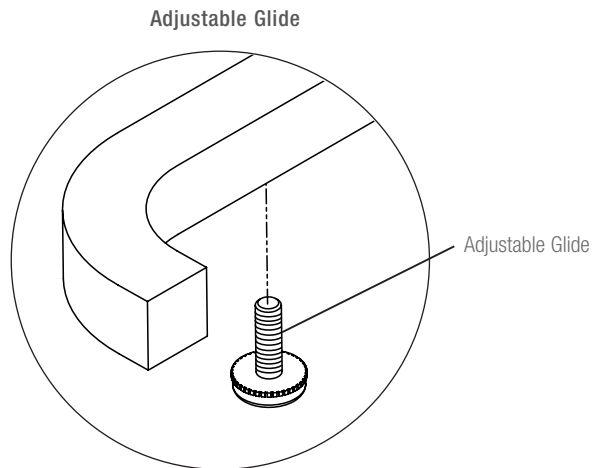Bronze Luxury-Lift® Power Recliner Battery Backup Assembly Instructions

⚠ WARNING – To reduce the risk of serious injury:

- In the event of a power failure, the battery backup provided may not lift the user to a fully raised position. To ensure lift functionality during a power outage, a battery backup system must be purchased separately. Contact your La-Z-Boy® dealer for more information. Replace the power supply batteries every six months and after each power failure use.

Battery Backup Assembly Instructions

The performance of the built-in battery backup cannot be guaranteed. If you require continuing use during a power outage, you should purchase a separate battery backup system. Contact your La-Z-Boy® dealer for more information.

The operating speed is reduced when operating on battery backup.

Always maintain constant, even pressure on the button when operating on battery backup to reduce energy consumption.

The built-in battery backup is located in the power supply and uses two 9-volt alkaline batteries (not included).

1. Unplug the power supply from the electrical outlet and disconnect the power supply connection at the base of the chair.
2. The battery compartment is located on the side of the power supply. Slide the battery cover up to access the battery compartment.
3. Place the connection caps over the battery terminals on each battery. Place the batteries in the compartment, taking care not to damage any wires.
4. Slide the battery cover over the compartment.
5. Proceed to the *Operating Instructions*.

Bronze Luxury-Lift® Power Recliner Power Supply Specifications:

Input: 100–240V.A.C. 50/60Hz. 3.0A

Output: 29V \equiv 2.0A

Bronze Luxury-Lift® Power Recliner Operating Instructions

⚠ DANGER – To reduce the risk of electric shock, serious injury or death:

- Do not let children play on or near the chair when in use. Serious injury or death could occur if head or body parts are under the chair when the chair is in motion.
- Liquids exposed to electricity pose an immediate risk of electric shock. Do not use this unit if the power control is damaged, wet, in contact with or near spilled water or any liquid. Do not use the unit if the button activation switch has been exposed to water or any liquid. Do not handle the plug or remote control with wet hands.
- Keep cord out of reach of small children, and do not use where it may be a trip hazard.

⚠ WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- Plug directly into a wall outlet. Do not use an extension cord, adapter or surge protector.
- Do not place power supply underneath the chair.
- The Bronze lift chairs are intended for a single occupant, not exceeding 300 pounds. Do not operate the chair with more than one occupant in or on any part of the chair.
- Sit down in the unit slowly. To avoid tip over, do not throw your weight against the back of the unit. Move the unit into or out of the reclining position slowly, making sure that no children or pets are in close proximity.
- To safely exit the unit and prevent a tripping hazard, bring the back upright and close the legrest.
- Unit is designed to ease exit from the chair. Take care to not slip out of chair when chair is in motion.
- Remove hand wand when not in use, and store out of reach of children.

NOTICE – To reduce the risk of product damage:

- After 2 minutes of continuous operation of the lift and recline features, allow the features to rest for 18 minutes.

Getting Started

To avoid product damage, the lift and recline features should not be operated continuously for extended periods of time. After 2 minutes of continuous operation, allow the lift and recline features to rest for 18 minutes.

Plug the power supply **directly** into a wall outlet. Place the power supply on the floor, behind the chair. Keep the cords and cables away from the base and mechanism. Do not place the power supply underneath the chair.

Bronze Luxury-Lift® Power Recliner Operation

This recliner features a simple, easy-to-use hand wand with only two buttons to operate the recliner. The top **LIFT** button moves the chair from the upright seated to lifted position and from the recline to upright seated position, while the bottom **RECLINE** button moves the chair from the lifted to upright seated position and the upright seated to recline position. Both the back and legrest move at the same time.

To sit down in the chair:

1. Stand beside the chair, press down on the **LIFT** button and hold. Release the button when the chair reaches a comfortable height to sit against, depending on your personal needs and preference.
2. Step back into the chair and sit in the center of the seat while keeping your feet firmly on the floor. Use the chair arms for support and sit slowly.
3. Press down on the **LIFT** button again and hold to lower the chair. Release the button when the chair reaches the upright, seated position.

Bronze Luxury-Lift® Power Recliner Operating Instructions

To recline the chair:

The back and legrest move together using the **LIFT** and **RECLINE** buttons on the front of the hand wand.

1. With the chair in the seated position, press and hold the **RECLINE** button to recline the back and raise the legrest. Release the button when you are in a comfortable position.
2. Make adjustments to the back and legrest recline as preferred.
3. To return the chair to the upright, seated position, press and hold down on the **LIFT** button. Release the button when you reach the upright, seated position.

To stand up from the chair:

The **LIFT** button provides a coordinated movement of the lift function.

1. Firmly position your feet on the floor to maintain balance and prevent sliding or falling. Press down on the **LIFT** button and hold to raise the chair to the standing position.
2. Release the button when the chair reaches a height that enables you to comfortably stand. Carefully exit the chair.
3. Return the chair to the upright, seated position if you will be away from the chair for an extended period. Press down on the **RECLINE** button to lower the chair. Release the button when the chair is in its upright, seated position.

Additional Operating Features:

Safety Legrest

The legrest of the Bronze lift chair closes using spring pressure for safety. When the path of a closing legrest is blocked by most ordinary obstructions, the obstacle in its path overcomes the spring pressure and stops the legrest from closing.

If the legrest stops moving when returning to the closed position, carefully look for an obstruction and remove it. Gently pull up on the legrest, remove the obstruction and gently release the legrest to allow it to close. If you cannot easily remove the object, ask for help or contact your La-Z-Boy® dealer.

Power Supply Over-Current Protection

The Bronze series lift chairs are equipped with over-current protection. This feature prevents damage to the motors and electronics if there is an overload, short circuit or ground fault. Should the system detect an over-current, the system shuts down and the recliner stops functioning.

If the over-current protection is activated, stop pressing the button in use. The system should reset automatically. If you believe the chair is not being overloaded, or experiencing a short circuit or ground fault, you may restart the function of the chair after the button has been released. Should the over-current protection trip repeatedly, refer to the Troubleshooting section in this manual.

Additionally, the power supply includes a green LED status light to indicate power is being supplied to the system. If the light is **not** illuminated, no power is being supplied to the system. Refer to the *Troubleshooting* section in this manual if the power supply is plugged in, but the green light is not illuminated.

Bronze Luxury-Lift® Power Recliner Operating Instructions

FRONT

Coordinated Lift & Return Button

When seated, press and hold down on the **LIFT** button to raise the chair to the standing position.

– OR –

When reclined, press and hold down on the **LIFT** button to return the back to the upright position and the legrest to the closed position.

Release the button to stop in any reclined position, or when fully upright.

Coordinated Lower & Recline Button

When lifted, press and hold down on the **RECLINE** button to lower the chair to the seated position.

– OR –

When seated, press and hold down the **RECLINE** button to recline the back and raise the legrest.

Release the button to stop in any reclined position, or when fully reclined.

(Page intentionally left blank)

⚠ DANGER

– To reduce the risk of burns, electric shock, serious injury or death:

- Liquids exposed to electricity pose an immediate risk of electric shock. Do not use this unit if the power control is damaged, wet, in contact with or near spilled water or any liquid. Do not use the unit if the button activation switch has been exposed to water or any liquid. Do not handle the plug or remote control with wet hands.

⚠ WARNING

– To reduce the risk of burns, fire, electric shock, serious injury or death:

- Remove hand wand when not in use, and store out of reach of children.

Refer to the *Hand Wand Removal Instructions* for directions on how to remove the hand wand.

Bronze Luxury-Lift® Power Recliner Troubleshooting

WARNING – To reduce the risk of serious injury:

- Do not attempt to disassemble or service the pump or motor; no serviceable parts are inside. Contact your La-Z-Boy® dealer for service.
- Check the condition of the power supplies, electrical cords, plugs, and connectors often. Keep the power supplies and electrical cords away from heat sources. Never operate the unit with a damaged power supply, electrical cord, plug or connector. Contact your La-Z-Boy® dealer for service.
- The Bronze lift chairs are intended for a single occupant, not exceeding 300 pounds. Do not operate the chair with more than one occupant in or on any part of the chair.

Troubleshooting for Lift and Recline Operation

If the recline or lift features do not work, check the following:

1. Make sure the 120-volt power cord is plugged into an electrical outlet.
2. Check the power supply to make sure the green LED power indicator light is on.
3. Make sure the electrical outlet is working. For example, make sure there are no blown fuses or tripped circuit breakers. If you are unsure of the outlet's condition, check with a qualified electrician or service technician.
4. Make sure all the connections are secure; refer to the *Assembly Instructions*.
5. Make sure the electrical cords are not crimped or pinched. Make sure the electric cord or plug is not damaged.
6. Make sure the operating capacity has not been exceeded and there is only one occupant in the chair; refer to the *Operating Instructions*.

Power Supply Replacement

If the power supply must be replaced:

1. Unplug the power supply from the electrical outlet.
2. Disconnect the 120-volt power cord from the power supply, and disconnect the power supply from the motor connector at the rear lower base of the unit. Refer to the *Assembly Instructions*.
3. Connect the new power supply to the motor connector at the rear lower base of the unit; align the motor connector to the power supply and push together until fully seated. Connect the 120-volt power cord to the new power supply; align the connector to the power supply and push together until fully seated. Refer to the *Assembly Instructions*.
4. Proceed to the *Battery Backup Assembly Instructions*.

(Page intentionally left blank)

Luxury-Lift® Hand Wand Removal Instructions

⚠ WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- Unplug the power cord before connecting or disconnecting the hand wand.

1. Unplug the 120 volt power cord from the electrical outlet.
2. Stand behind the chair and disconnect the power supply from the motor connector at the rear lower base of the chair (FIGURE 1).
3. From the front of the chair, locate the hand wand connection on the seat. Open the locking cover and gently pull the connection apart to separate the hand wand from the body (FIGURE 2). Store the hand wand out of reach of children.

FIGURE 1


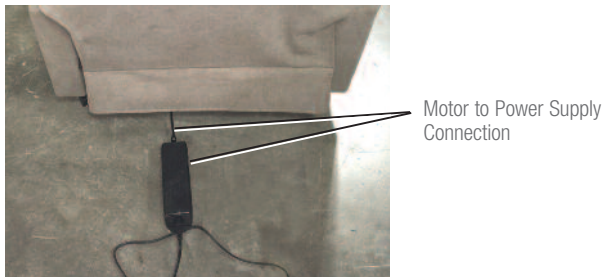
FIGURE 2

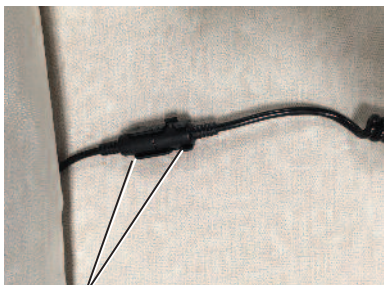Hand Wand Connection with Locking Cover

Luxury-Lift® Hand Wand Installation Instructions

⚠ WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- Unplug the power cord before connecting or disconnecting the hand wand.

1. Unplug the 120-volt power cord from the electrical outlet.
2. Stand behind the chair and disconnect the power supply from the motor connector at the rear lower base of the chair (FIGURE 1).
3. From the front of the chair, locate the hand wand body connector near the back of the seat. Align the arrow on the hand wand connector to the arrow on the body connector and push together until fully seated (FIGURE 2). Close the locking cover to secure the connection. (FIGURE 3).
4. Connect the power supply to the chair.

⚠ WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- Plug directly into a wall outlet. Do not use an extension cord, adapter or surge protector.

5. Plug the 120-volt power cord **directly** into the electrical outlet.

FIGURE 1


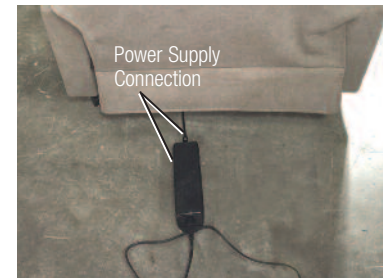
FIGURE 2

FIGURE 3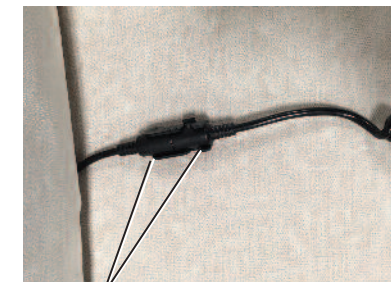

Hand Wand Connection Secured with Locking Cover Closed

Moving the Bronze Lift Chair


⚠ WARNING – To reduce the risk of burns, fire, electric shock, serious injury or death:

- Before moving the chair, disconnect the electrical plug from the wall outlet. Moving the chair requires two people, one on each side of the chair lifting the frame near the base. Secure the mechanism as shown to prevent unexpected opening or movement. Only move the product by the lower frame, never by the legrest, arms or reclining back.

Moving the Bronze lift chair requires two people, one on each side of the chair. The back can be removed for handling ease; refer to step 6 of the *Assembly Instructions*, for directions on back removal.

1. Unplug the power supply from the electrical outlet.
2. Disconnect the 120-volt power cord from the power supply, and disconnect the power supply from the motor connector at the base of the unit. Refer to the *Assembly Instructions*.
3. Secure hand wand by placing it between the seat and arm.
4. Secure the mechanism with a cable tie to reduce the chance for movement. Working from behind the recliner, carefully loosen the fastening tape securing the back tailgate cover, lift the cover and secure the mechanism by attaching a cable tie around the two lower rear bottom rails as shown below.
5. Have two people stand to the side of the chair (one on each side) and grip the lower arm frame to lift and move the chair.

Important: If a hand truck is used, only use from the back of the chair, never from the front of the chair.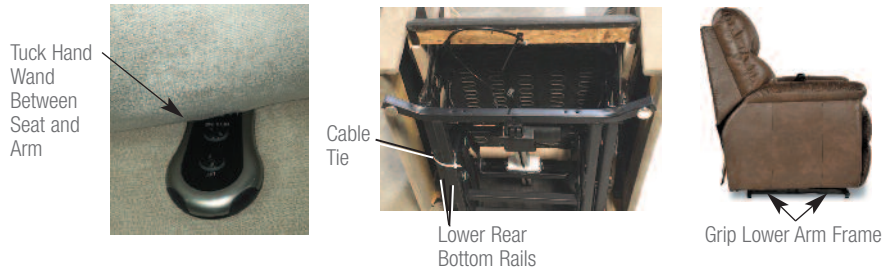


Furniture Placement

⚠ WARNING – To reduce the risk of serious injury:

- Provide a clear path for operation of the back and legrest. Place tables and area rugs at a distance to allow the legrest to fully extend without rubbing or interference.

To place your reclining unit the proper distance from the wall, fully recline the unit and leave a minimum of 2" of space between the top of the back and the wall.

Place tables and area rugs at a distance to allow the legrest to fully extend without rubbing or interference.

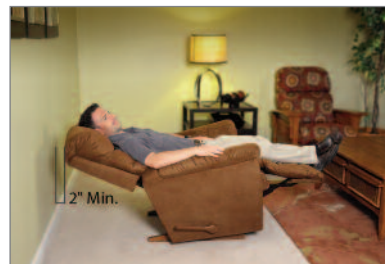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